




ImageMaster_USB Camera setting in AMCap: “After AMCap Installed”

1. Create AmCap Images saving folder:

Right click mouse anywhere on desktop → New → Folder and type “Camera Images”



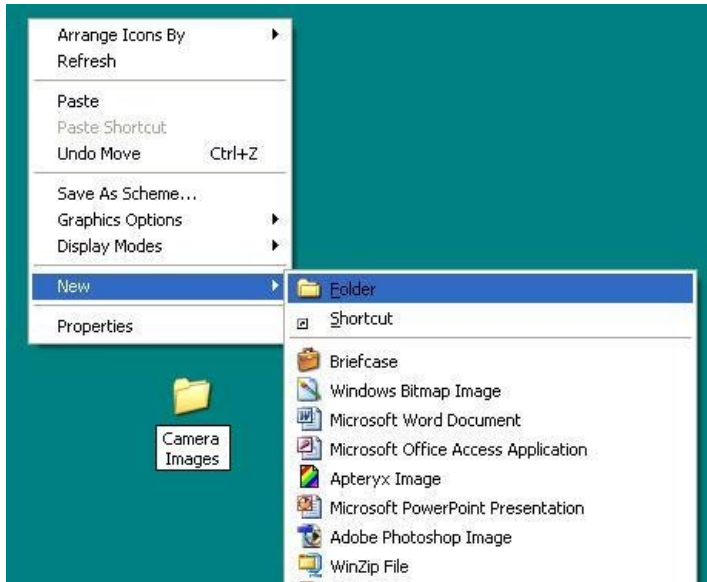


Then open AMCap and go to Capture → Setup → Still Capture → Browse → Desktop → select Camera Images

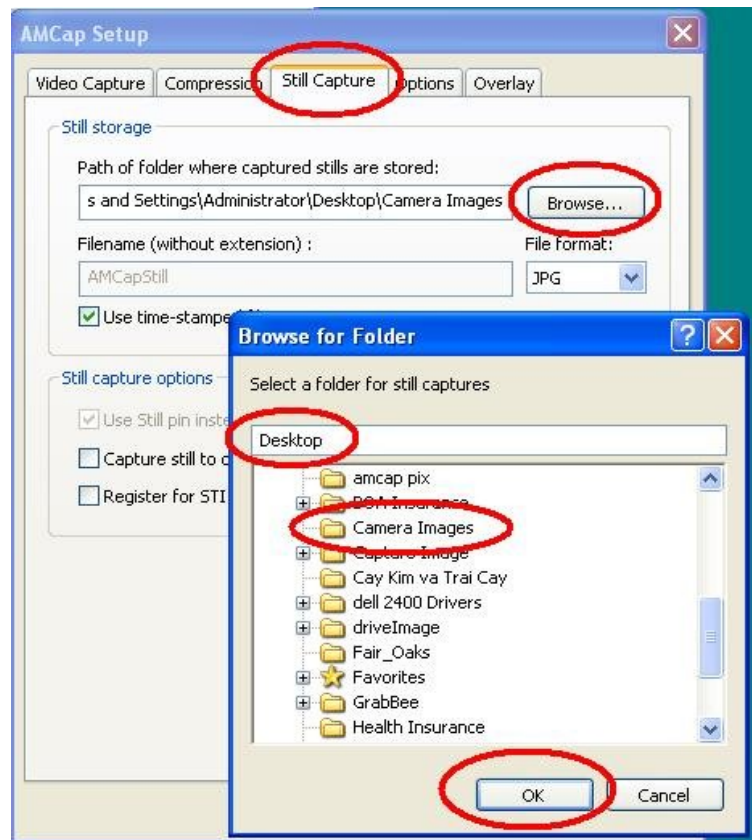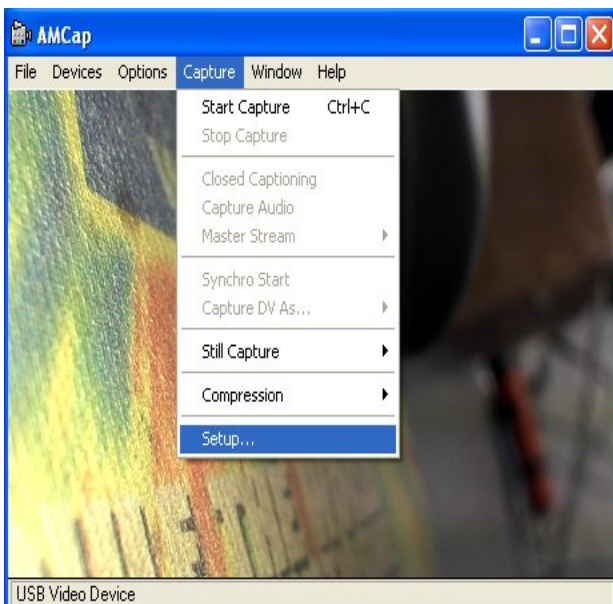


2. Color Adjust in AmCap:

Go to Capture → Setup → Options and Uncheck boxes “Remember images controls” & “Remember camera controls”

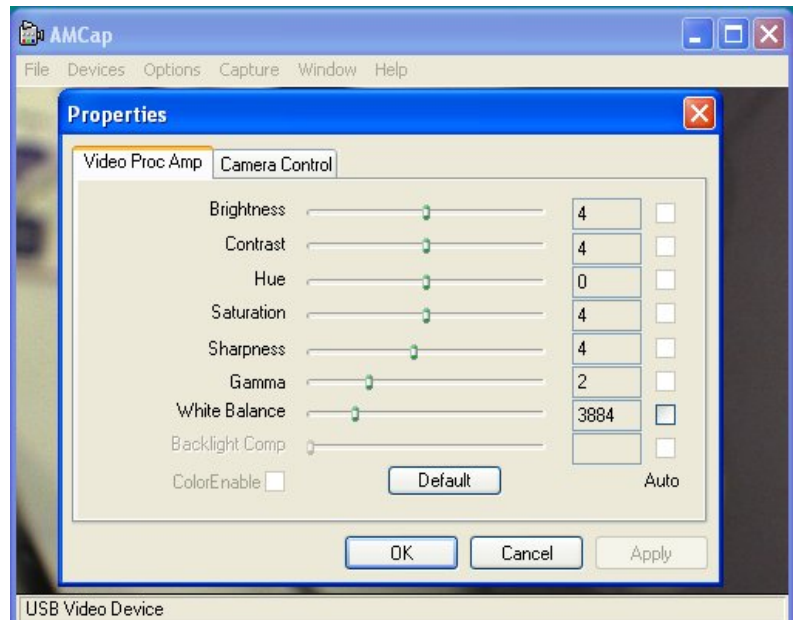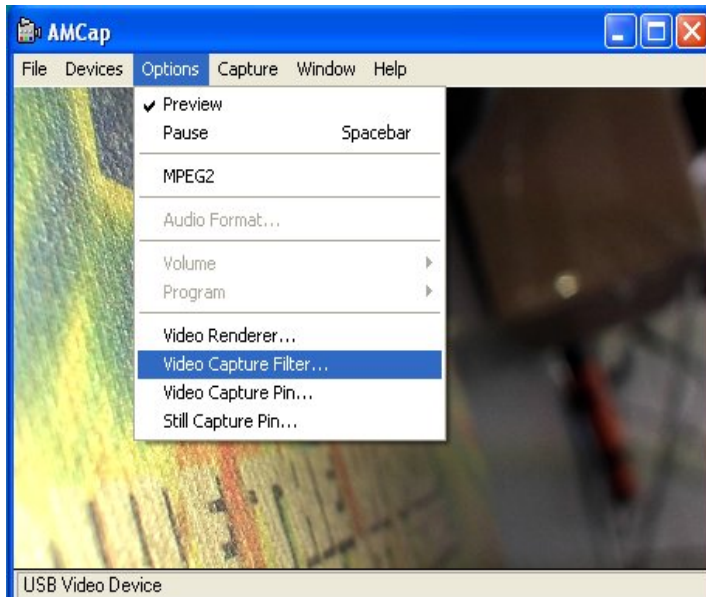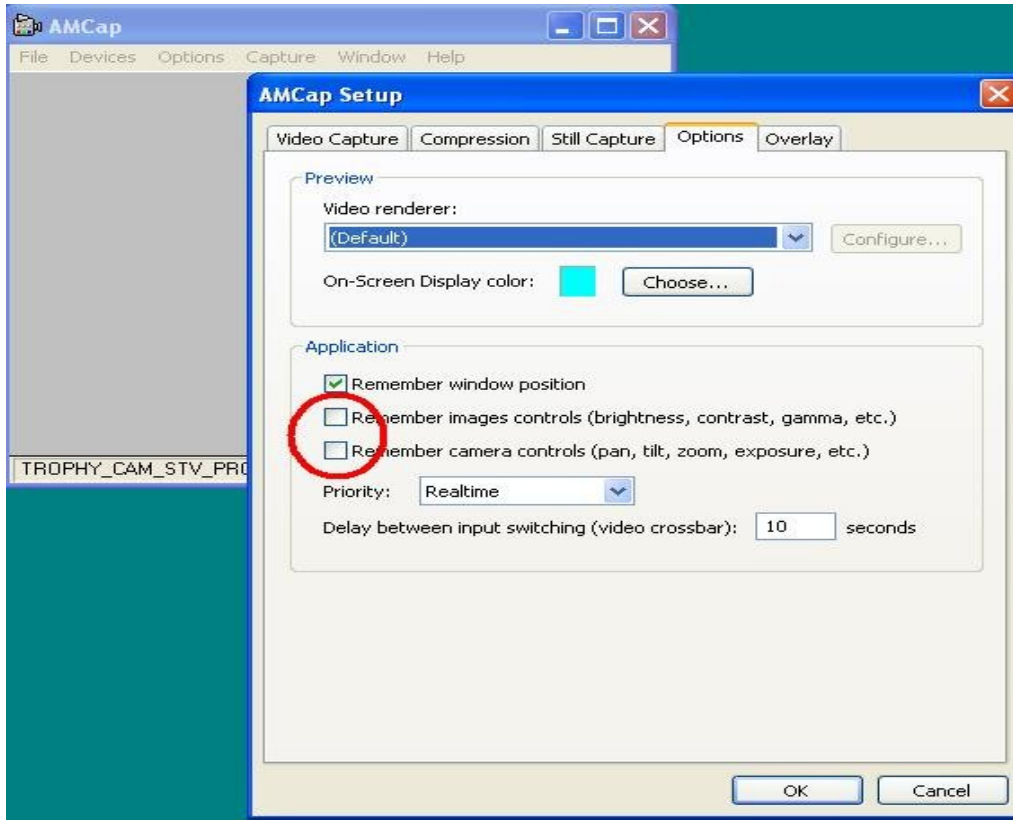

3. Am_Cap undetecting camera:

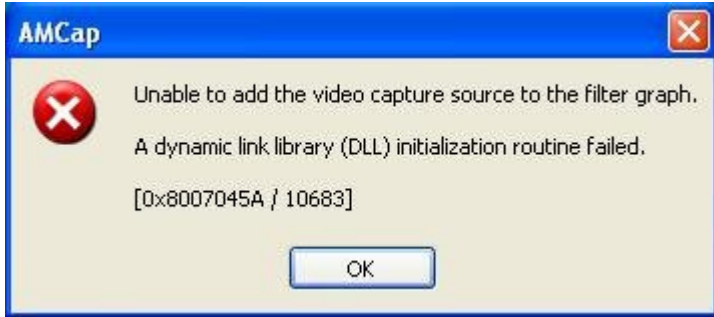

- A. Check to make sure USB cable connect secure in both end of computer and camera hand piece.
- B. Select Devices: **"USB Video Device"** in Windows Xp or **"ImageMaster USB"** in Windows 7
- C. Close AMCap and re-open.

