

## ImageMaster\_USB Camera Setting In DentalEye

After Install DentalEye Imaging & VideoPluginSetup, then completed **Step 1** to **Step 4**

The screenshot shows the DentalEye 3.1.1258 application window with the 'Video Plugin setup.' dialog box open. The dialog box contains the following settings:

- Video device:** Camera using UVC hardware triggering (circled in red, labeled Step 4)
- Info for selected device:** For UVC standard cameras with hardware triggering utilizing the default drivers of the OS. This includes the Imagin ImageMaster
- Show "Captured images" window:** Checked (circled in red, labeled Step 2 and Check Box)
- Camera control:** Control device: None
- Connected to serial port:** Com 1
- Trigger options:** Trigger when released (selected)

Four red arrows indicate the steps:

- Step 1:** Points to the 'SETUP' icon in the top toolbar.
- Step 2:** Points to the 'Show "Captured images" window' checkbox.
- Step 3:** Points to the 'OK' button.
- Step 4:** Points to the 'Video device' dropdown menu.