


FOCUS DETAIL AND POSITIONS

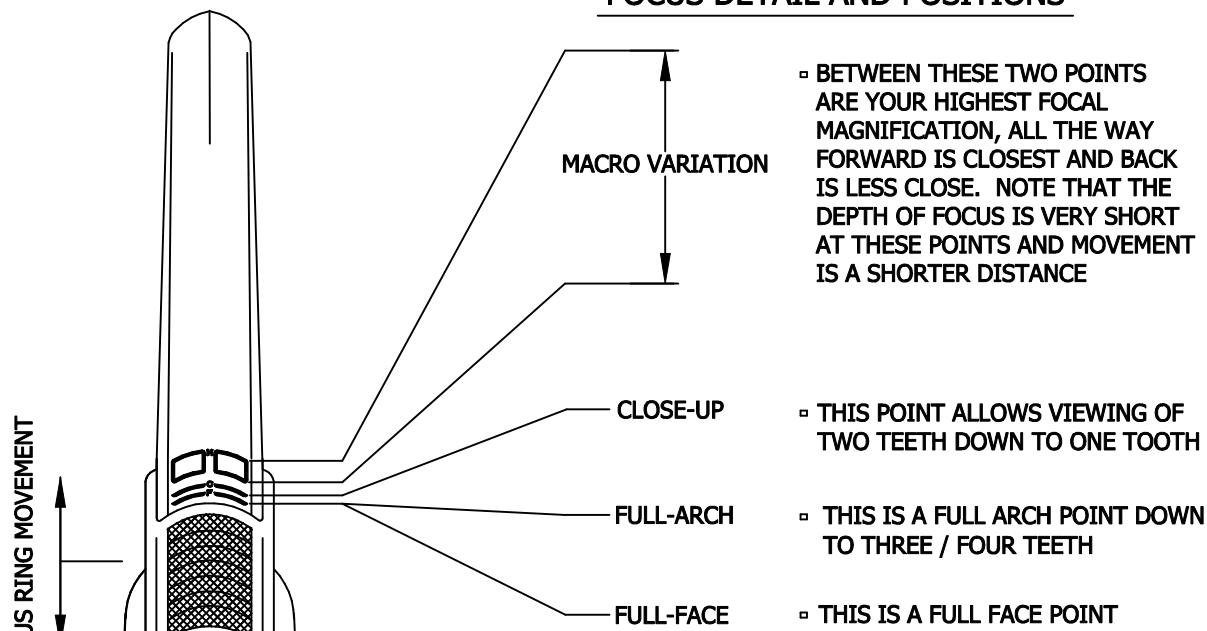

ON BOARD BUTTON CONTROLS

- CAPTURE BUTTONS, LOCATED ON BOTH SIDES
- LIGHT CONTROL, THREE LEVELS OF BRIGHTNESS
 - 1) NORMAL INTENSITY, DEFAULT SETTING
 - 2) LIGHT ON, LOWER INTENSITY
 - 3) LIGHT ON, HIGH INTENSITY (BRIGHTEST SETTING)
 - 4) LIGHT ON/OFF
- COLOR MODE BUTTON, TOGGLE BUTTON TO GET: BLACK AND WHITE (GREY SCALE) DIAGNOSTIC MODE (CARIES AND TRANSLUCIENCE)
- ON SCREEN ARROW BUTTON: PUSH ON, PUSH OFF
IMAGE ORIENTATION: PUSH/HOLD TO MIRRORED/DIRECT (HOLD BUTTON FOR MORE THAN 2 SECONDS)